Sothys Micro Dermobooster™.

- Sothys Micro Dermobooster™ behandlingsprotokoll p. 2 to 5
- Professionella produkter p. 6
- Behandlingsförberedelse p. 7 & 8
- **Beauty diagnosis** Sothys Micro Dermobooster™ treatment p. 9
- **Informationsmemo** Sothys Micro Dermobooster™ treatment p. 10
- **Follow-up during** the Sothys Micro Dermobooster™ treatment p. 11

Sothys Micro Dermobooster™.

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TM Treatment preparation

RECOMMENDATION OF HYGIENE PRODUCTS

- **Disinfectant detergent:** cleans and disinfects surfaces and furniture as well as the accessories that will be in contact with the client.
- **Soap:** to wash hands
- **Hydroalcoholic product:** antiseptic which allows to sanitize the hands.

CLEANING AND STERILIZING THE HANDS

1. Cleaning the hands:

01. Wet the hands and forearms using warm water

02. Apply a dose of soap to the palm of your hands

03. Rub your hands together for at least 30 seconds to create foam

04. Soap the forearms well

05. Rub the backs of the hands and the spaces between the fingers

06. Clean the nails in the palm of your hand

07. Soap the wrists

08. Rinse thoroughly under running water

09. Dry the hands and the forearms using a disposable paper towel

10. Turn off the tap using the paper towel

CLEANING AND STERILIZING THE HANDS

2. Sterilizing the hands using an hydroalcoholic product

01. Pump once or twice into the palm of the hand without adding water

02. Spread the product across the surface of the hands for 15 seconds

03. Rub the back of each hand using the palm of the other hand, interweaving the fingers

04. Rub the tips of the fingers on each hand in the palm of the other hand

05. Continue rubbing until the hands are dry. Do not wipe the hands.

Beauty diagnosis

Micro Dermobooster™ protocol

CONTACT INFORMATION(1)

Surname: _____ First name: _____ Date of birth: _____
 Tel: _____ Email: _____

SKIN CHARACTERISTICS

- Skin type: Dry Combination Oily
- Sensitivity: slight sensitivity, please specify the area: _____ Very sensitive** Reactive**
- Any evidence of couperosis (redness, visible blood vessels): yes no, if yes, please specify the area to be excluded: _____
- Any evidence of rosacea (redness, visible blood vessels, papulopustular lesions): yes no, if yes, please specify the area to be excluded: _____
- Any evidence of angioma: yes no, if yes, please specify the area to be excluded: _____
- Thin skin: yes* no

ADDITIONAL INFORMATION

- Overall condition: Good Tired / Stressed, specify intensity on a scale of 1 to 10: _____
- Pregnant : yes* no
- Skin allergies: yes no → if yes please specify _____
- Dermatoses on the face: yes* no → if yes please specify _____
- Prone to cold sores on the mouth: yes no
- Ongoing dermatological medical treatment (Preparation or drug with vitamin A, vitamin C preparation(ascorbic acid), azelaic acid, cortisone cream): yes* no → if yes please specify _____
- Medical treatment: Anti-inf lammatory2** Anti-coagulant3**

RECENT EXPOSURE THAT MAY PREVENT PROTOCOL

- Have you waxed/shaved the area of skin to be treated in the last 24 hours? yes no
- Have you exposed the area of skin to be treated to intense UV light or the sun in the last 24 hours? yes no
- Have you exposed the area of skin to be treated to extreme weather conditions (cold, wind, sun, etc.) in the last 24 hours? yes no
- On the area of skin to be treated, have you had:
 - A home peel treatment in the last 24 hours? yes no
 - An aesthetic peel treatment in the last week? yes no
 - A medical peel treatment in the last month? yes* no
- Recent surgeries (in the last month) in the treatment area: injections, surgical interventions, implants... yes** no

(1)Personal data:

The data collected for this diagnosis will be processed by the beauty institute whose stamp appears below, and has been put in place in order to facilitate the progress and monitoring of the Micro Dermobooster™ machine protocol. The information will be kept for the duration of these processes, or as specifically indicated in applicable regulations. In accordance with applicable regulations, you have the right to access, amend, oppose, delete or transfer any of your personal data. You can exercise this right by contacting the beauty institute whose stamp is shown below. You can also at any time withdraw your consent to the collection and processing of your personal data.

*** By completing this skin diagnosis, you agree that the information collected will be used by your beautician specifically in this context.

INSTITUTE
STAMP

In: _____
 On: _____
 Signature: _____

(2) Examples: aspirin (Kardegic®), ibuprofen
 (3) Examples: acenocoumarol (Sintrom®); flindione (Previscan®); warfarin (Coumadine®); Rivaroxaban (Xarelto®); Dabigatran (Pradaxa®); heparin and derivatives (type Lovenox®, Innohep®)
 * Please seek medical advice ** Do not perform the treatment *** please check the box to give your consent

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DATE OF PROTOCOL 1:

Skin condition during the session:
 Normal
 Redness → areas
 Burning sensations → areas
 Tingling → areas
 Other, please give details:
→ Areas

How the client felt during the session:
 Good Sensitive

Any more comments:

If necessary, please note any changes to be made during the next session:

Medical advice to be asked for:
 Yes No

Products recommended for use at home after the protocol:

DATE OF PROTOCOL 2:

Skin condition during the session:
 Normal
 Redness → areas
 Burning sensations → areas
 Tingling → areas
 Other, please give details:
→ Areas

How the client felt during the session:
 Good Sensitive

Any more comments:

If necessary, please note any changes to be made during the next session:

Medical advice to be asked for:
 Yes No

Products recommended for use at home if different from protocol 1:

DATE OF PROTOCOL 3:

Skin condition during the session:
 Normal
 Redness → areas
 Burning sensations → areas
 Tingling → areas
 Other, please give details:
→ Areas

How the client felt during the session:
 Good Sensitive

Any more comments:

If necessary, please note any changes to be made during the next session:

Medical advice to be asked for:
 Yes No

Products recommended for use at home after the protocol: